

SaeboGlove | Clinical & Measurement Guidelines

1. Clinical Appropriateness

Indications for Use:

1. Neurological conditions including but not limited to stroke, brain injury, spinal cord injury, MS, GBS, ALS, and muscular dystrophy.
2. Brachial plexus injury
3. Radial nerve injury
4. Individuals exhibiting wrist drop and/or lacking finger extension.


Contraindications:

1. Not for use with clients exhibiting increased tone/spasticity.
2. Not for use with severe contractures or joint deformities in the fingers.
3. Not for use over open wounds and infected areas.
4. Not for use with severe edema.

2. How to Measure

To maximize comfort and fit, there are 4 measurements required (2 for the Wrist Splint and 2 for the Glove Liner). The four measurements will indicate the best possible size recommendation. Once you have determined the below measurements, use the chart to the right to determine the appropriate recommended size. Enter the final Wrist Splint and Glove Liner measurements on page 1.

STEP 1: WRIST SPLINT MEASUREMENTS:

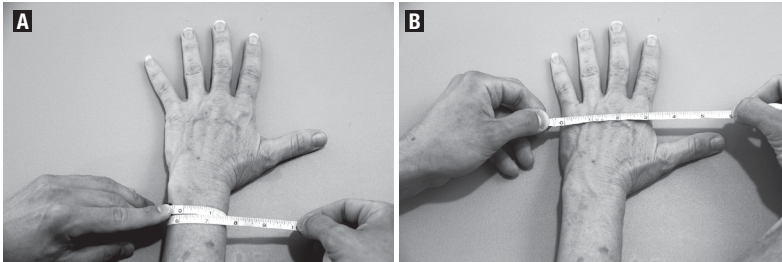


- A.) **Wrist Circumference:** measure wrist circumference just proximal to head of ulna: _____
- B.) **Dorsal Hand Width:** measure dorsal hand width just proximal to MCP's (do not wrap tape down lateral or medial side of hand): _____

		(A) WRIST CIRCUMFERENCE					
		4-4.5	4.5-5.5	5.5-6.5	6.5-7.5	7.5-8.5	>8.5
(B) DORSAL HAND WIDTH	<2.5	S	S	S			
	2.5-3.0	S	S	M	M		
	3.0-3.5	S	M	M	M	M	L
	3.5-4.0			M	M	L	L
	>4.0				L	L	L

ENTER FINAL MEASUREMENT ON PAGE 1

STEP 2: GLOVE LINER MEASUREMENTS:

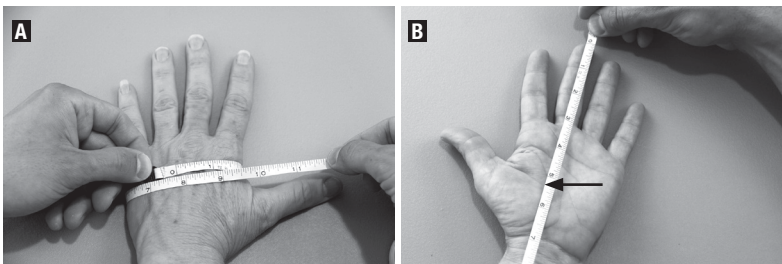

- A.) **Hand Circumference:** measure hand circumference by wrapping tape circumferentially around hand: _____
- B.) **3rd Finger Length:** measure from tip of 3rd digit to the most distal wrist crease: _____

		(A) HAND CIRCUMFERENCE					
		4-5.5	5.5-6.5	6.5-7.5	7.5-8.5	8.5-9.5	9.5-10.5
(B) 3RD FINGER LENGTH	6.0-6.5	XS	XS	XS			
	6.5-7.0	XS	XS	XS	S		
	7.0-7.5		S	S	S	M	
	7.5-8.0			M	M	L	XL
	>8.0				L	XL	XL

ENTER FINAL MEASUREMENT ON PAGE 1

Measuring Tips

1. If your Wrist Splint measurement corresponds with 2 different sizes, review the Glove Liner size as a reference. For example, if the Wrist Splint measurement corresponds with a Small and a Medium and the Glove Liner size is a Small, then consider downsizing to a Small Wrist Splint.
2. If the Glove Liner measurement corresponds with 2 different sizes, select the smaller size to assure a proper snug fit.